

## September 2020 Application update- BrailleNote Touch Plus Release notes

### KeyWord and KeyFiles now supports PDF

- BrailleNote Touch Plus now supports the opening of text PDF files using the native KeyWord application.
- When selecting PDF files, you are now given the option to open directly with KeyWord.
- KeyWord will extract all the text from a PDF file, providing it's a true text PDF.
- No formatting is attained when opening PDF Files.

### KeyMail

- **oAuth support for access to Gmail**
  - When adding a Gmail account, a Sign in with Google is available.
  - No need to enable less secure apps.
  - Security Password for third-party apps no longer required.
  - KeyMail is an approved Google email client.

### Fixes and Enhancements:

- Fixed issue where parentheses were computing wrongly on specific fraction calculations.
- Uploaded the latest onboard Spanish User guide.