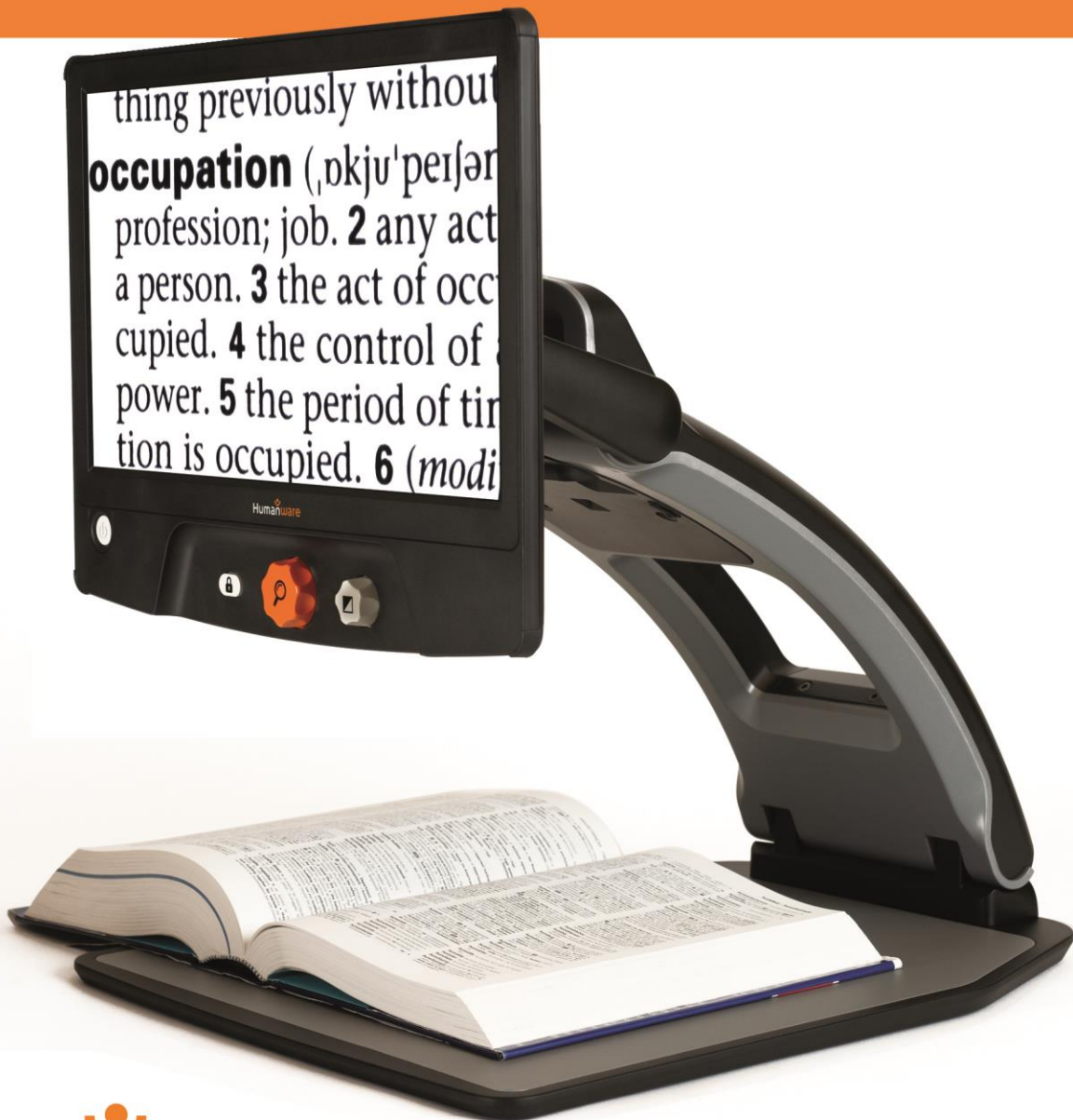
Reveal 16

User Guide Guide d'utilisation Guía del usuario

Contents

Introduction	1
Getting to Know your Reveal	1
Box Content.....	1
Setting Up.....	2
Overview of your Reveal	5
Using your Reveal.....	8
Turning on your Reveal	8
Using the Reveal for Desktop viewing	8
Using the Reveal for Distance Viewing	9
Zooming In and Out	10
Changing Contrast.....	10
Adjusting the Contrast Ratio	11
Autofocus lock.....	11
Menu	12
Stowing your reveal	14
Line and blinds	15
Adding a line or blinds.....	15
Adjusting a line or blinds.....	15
Using Reveal on another Screen	16
Troubleshooting.....	16
Safety and Maintenance	17
Specifications	18
FCC Info	18
Warranty	19

INTRODUCTION

Congratulations on purchasing the Reveal 16, the complete foldable and transportable magnification system with the best image quality on the market. With its simple and intuitive interface, you will be back to doing the things you love in no time.

Carefully read the operating and safety instructions described in this user guide prior to using your Reveal. If you have any questions about your system, please contact HumanWare Customer Service by consulting the contact information in the back of this user guide.

Copyright 2019 by Technologies HumanWare Inc., all rights reserved. No part of this publication may be reproduced or distributed in any form or by any means, or stored in a database or retrieval system, without the prior written permission of Technologies HumanWare Inc.

GETTING TO KNOW YOUR REVEAL

This section provides basic information to help you familiarize yourself with your Reveal.

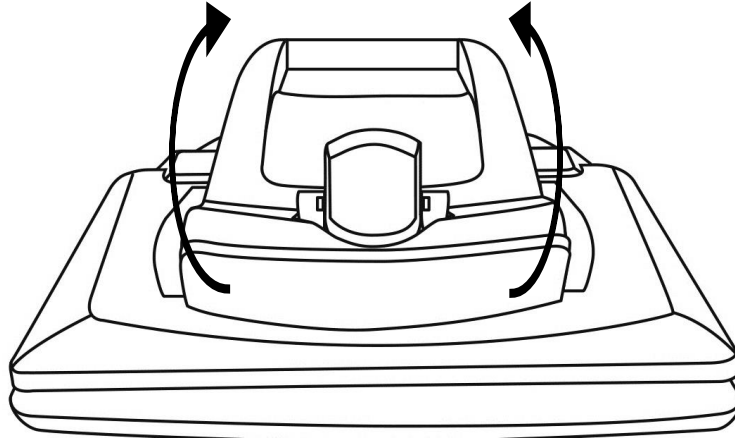
BOX CONTENT

- Reveal CCTV
- Printed User Guide
- Getting Started Sheet
- Power Adapter
- North American Power Cable
- European Power Cable
- UK Power Cable
- Australia Power Cable


SETTING UP


The Reveal was designed with simplicity in mind. Follow the steps below to quickly get your device up and running.

1. Carefully remove your Reveal from its box. It will be folded in its carrying position.
2. Place your Reveal on a flat and stable surface, such as a table, while making sure the HumanWare Logo is upside down in front of you.
3. With one hand, hold the reading tray down onto the table, and with your other hand, lift up the top of the bridge (where the camera is located). The bridge should lock into place in its open position. This may require a bit of force.

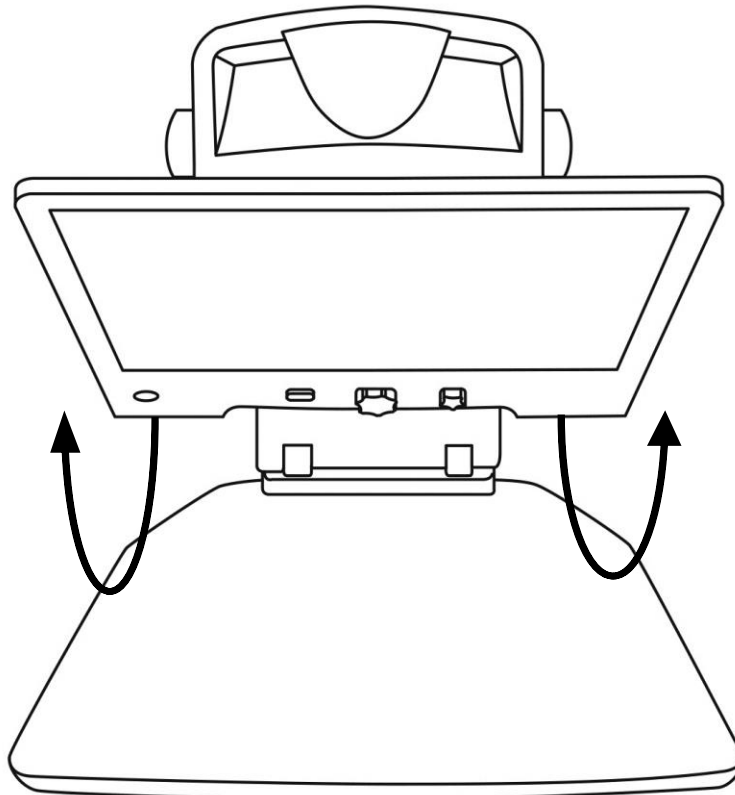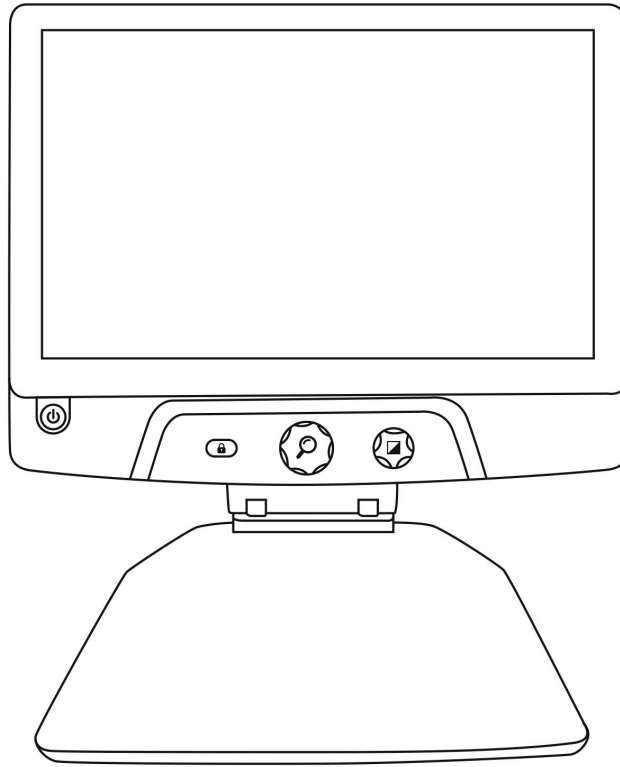
4. Carefully hold the screen by the sides and pull the bottom of the screen towards you until you reach a comfortable viewing position.

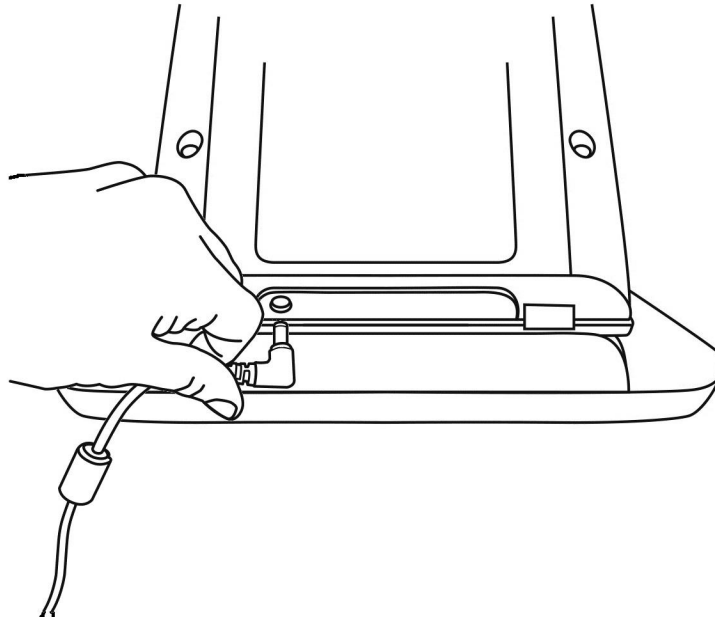
5. There will be multiple cables in your box
 - A universal power adapter cable
 - Country-specific power cables
6. Plug the power cable into the back of your Reveal.

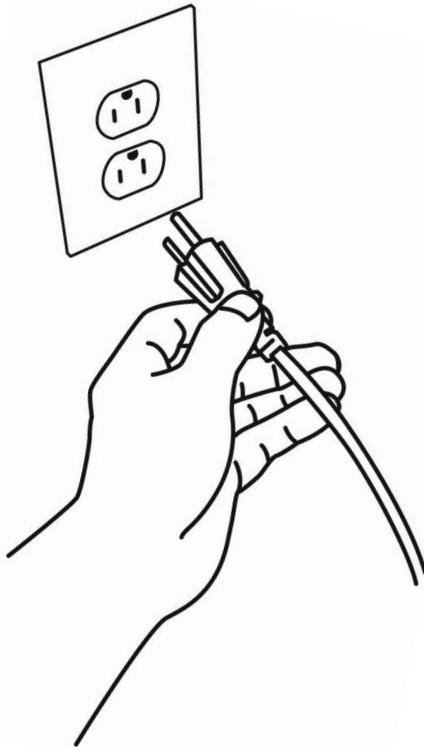
7. Plug the power cable into a power outlet.

Note: Your Reveal's Power button should now light up red meaning your device is powered and ready to use. If you have an optional battery installed, the light will be orange meaning your battery is recharging.

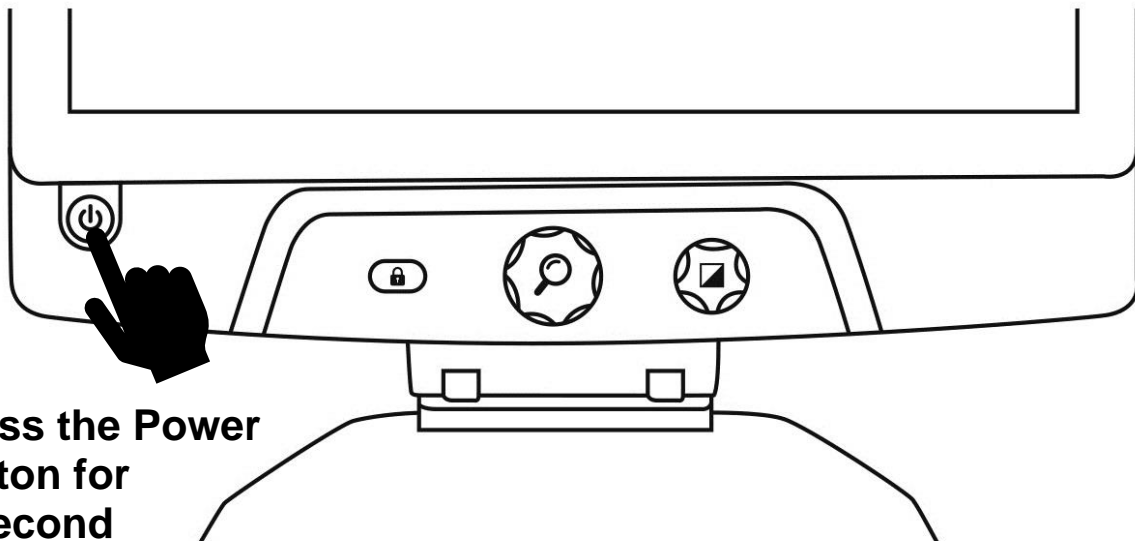
8. Press and hold the red Power button for 1 second and your Reveal will open immediately.

**Press the Power
button for
1 second**


OVERVIEW OF YOUR REVEAL

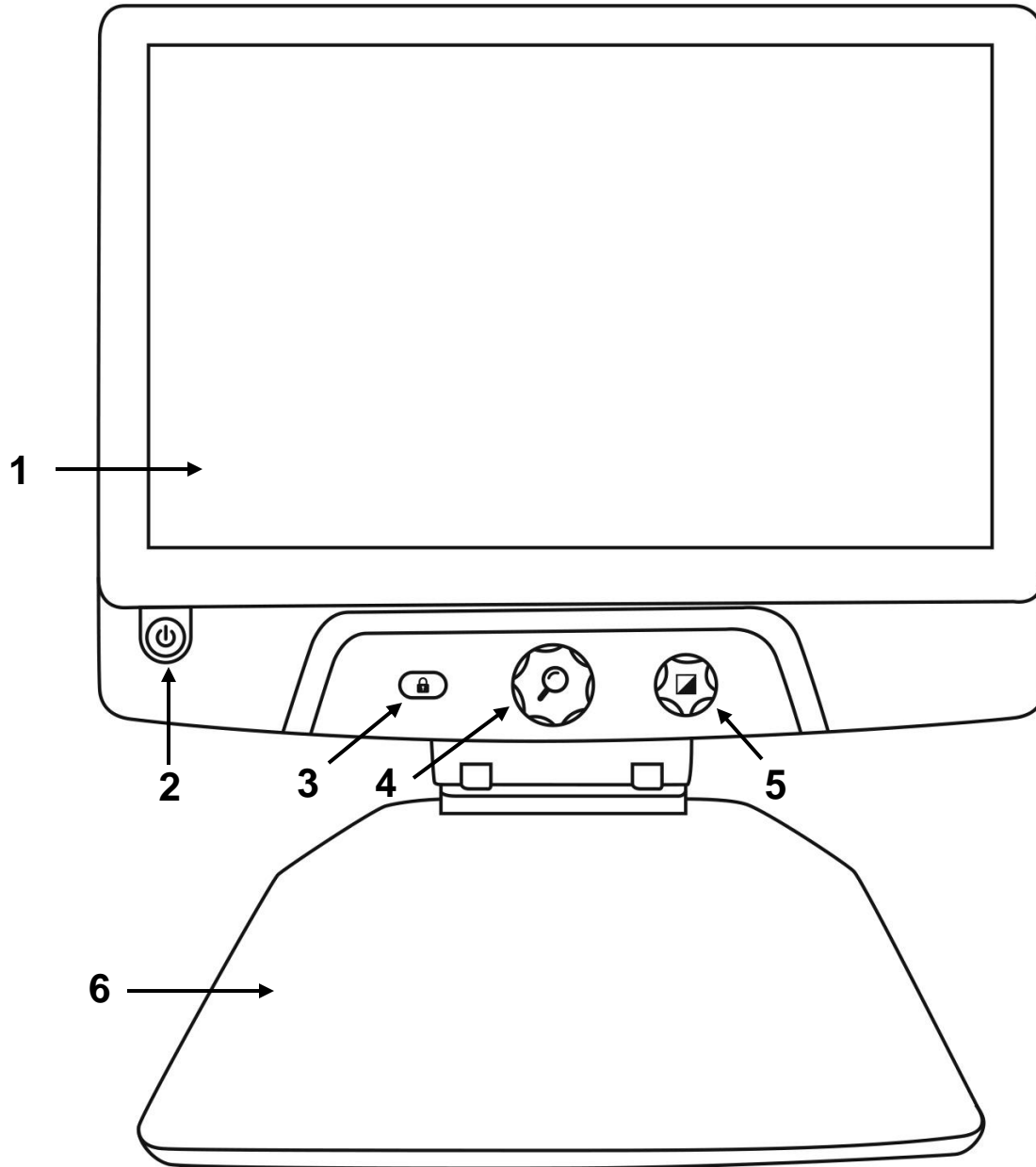Front


1. Screen
2. Power button
3. Autofocus Lock button
4. Zoom button
5. Contrast button
6. Reading tray

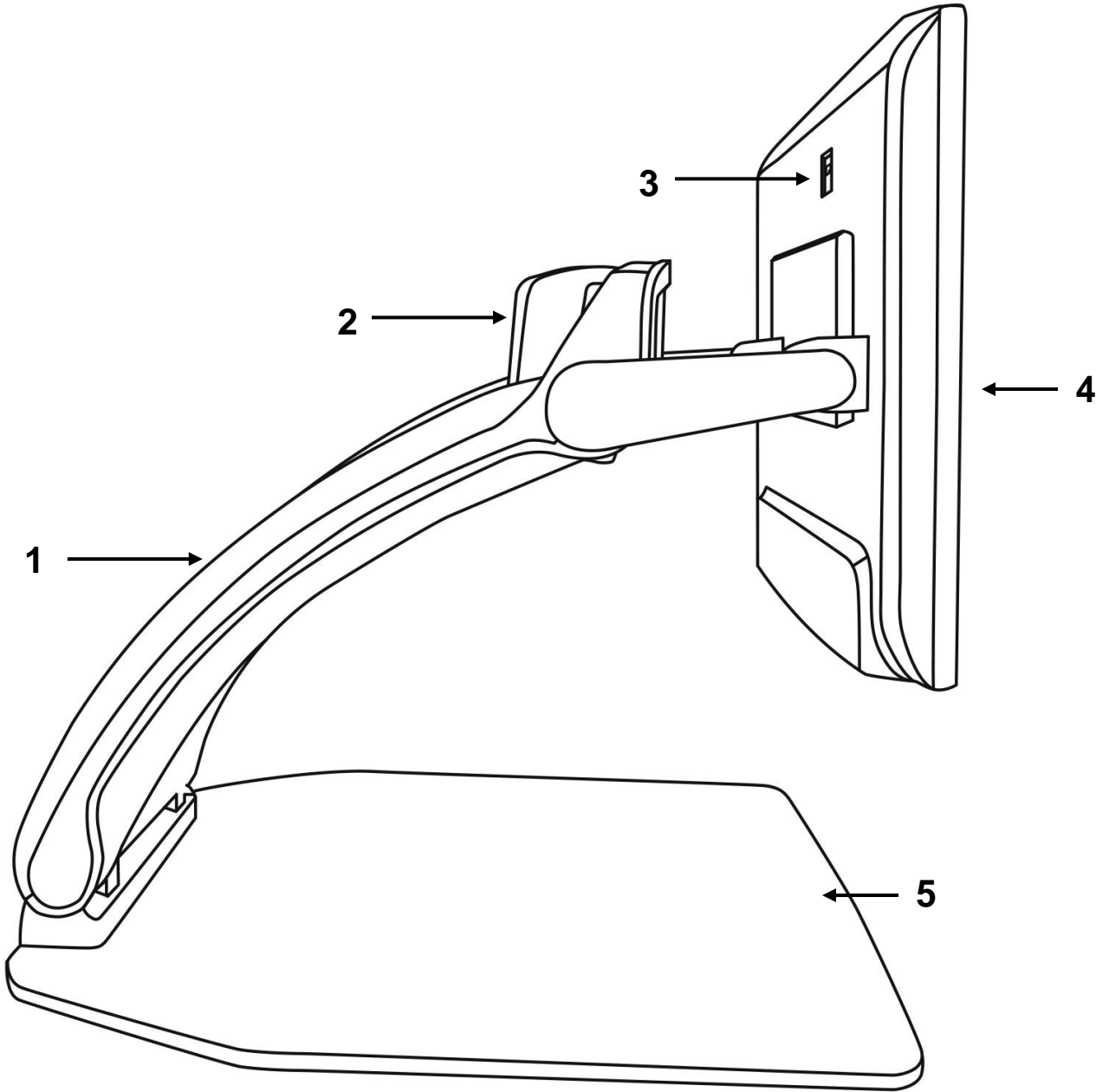Side

1. Bridge
2. Rotatable camera
3. High-Resolution video port
4. Screen
5. Reading tray

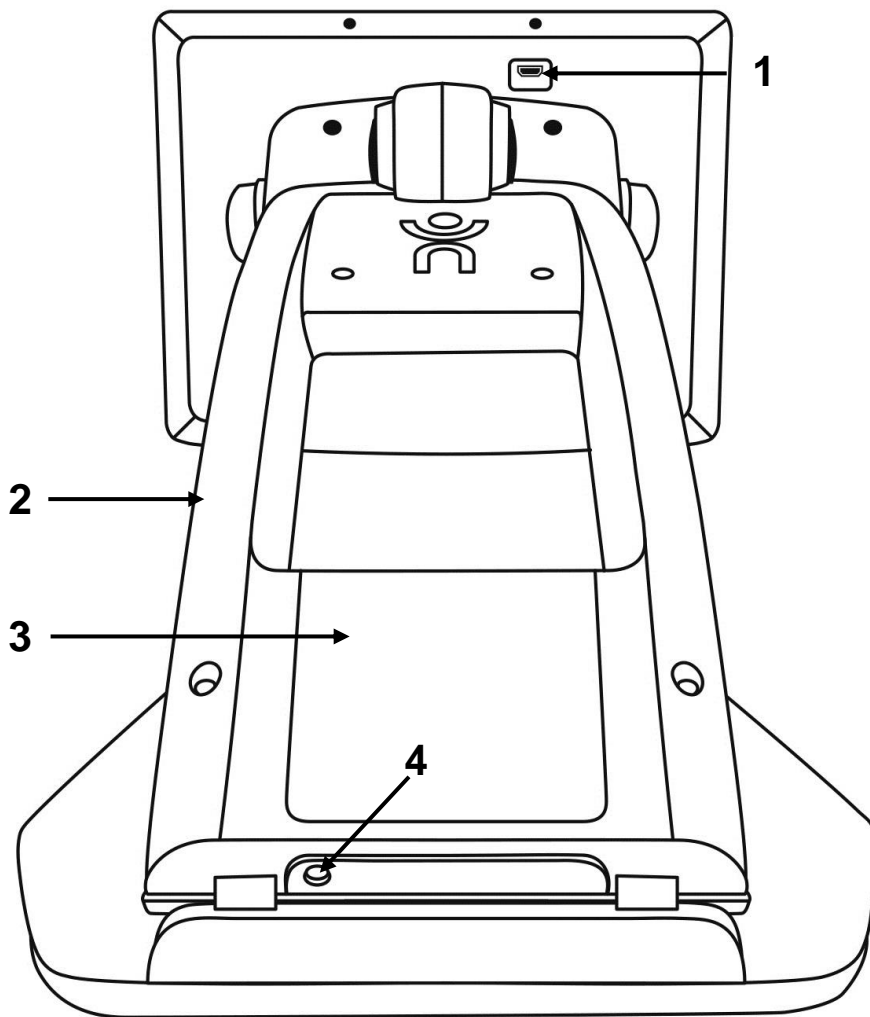Back

1. High-Resolution video port
2. Bridge
3. Battery (optional)
4. Power connector

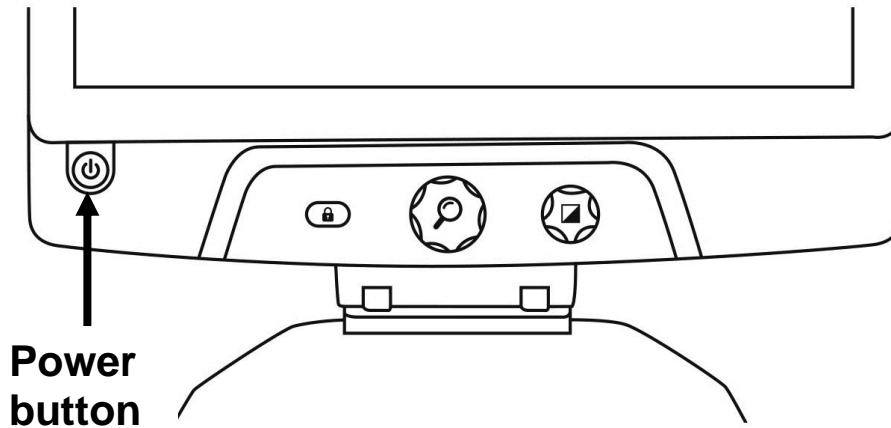
USING YOUR REVEAL

TURNING ON YOUR REVEAL

When your Reveal is off, the circular Power button on the bottom-left side of your screen will glow red, or orange if you have an optional battery installed that is charging.


To turn on your Reveal, simply press the Power button for 1 second.

The screen will turn on and the button light will change from red/orange to green. If the Power button is neither red, orange, nor green, make sure your Reveal is properly plugged in a power outlet.

USING THE REVEAL FOR DESKTOP VIEWING

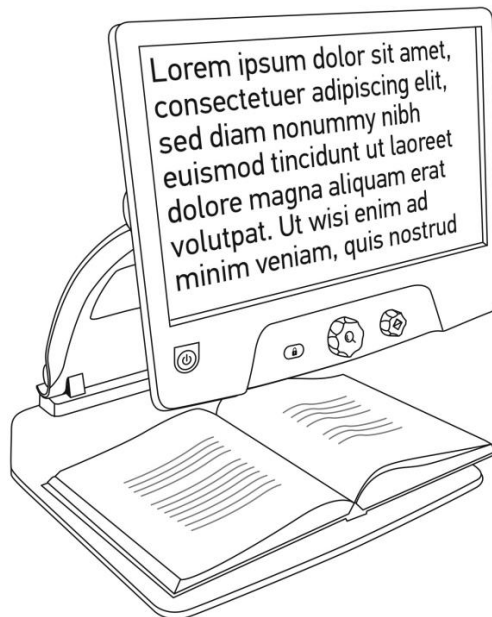
Upon opening your Reveal, you will be in Desktop viewing mode. In Desktop mode, everything placed on the reading tray will be shown in real time on the 16-inch screen. This is the primary way of using the device.

USING THE REVEAL FOR DISTANCE VIEWING

Your Reveal can also be used to view distant objects. To do so, simply rotate the camera located on top of your device.

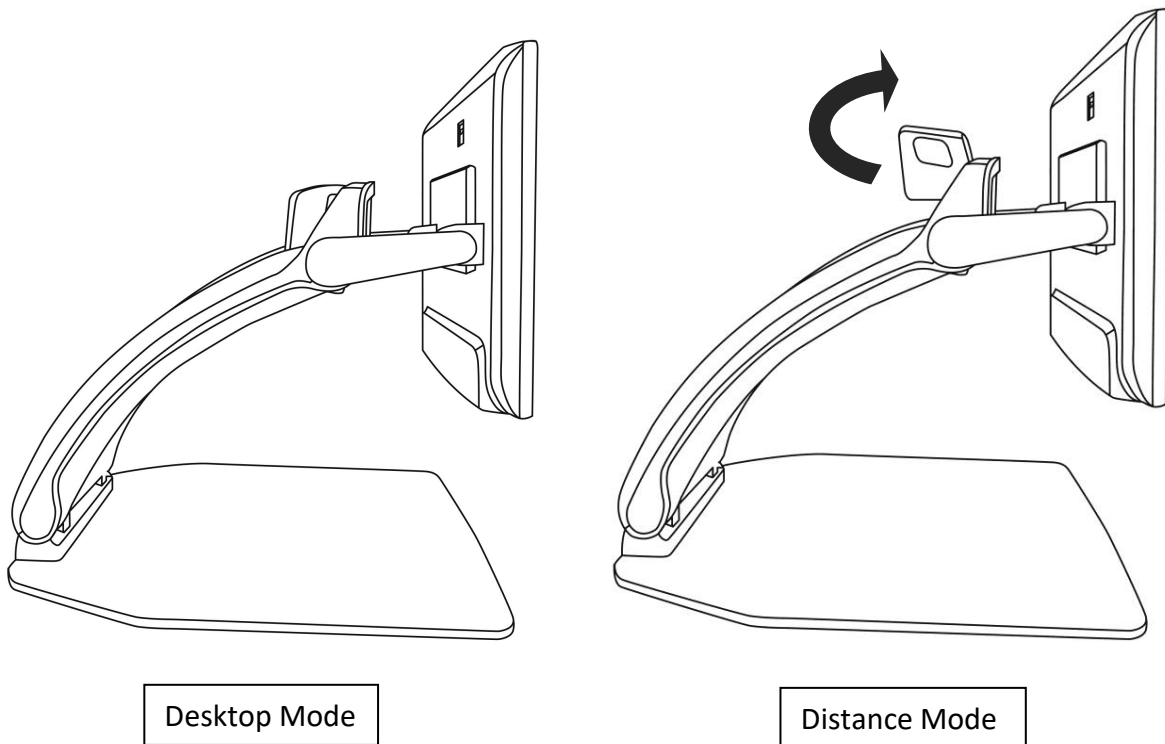1. Locate the camera on top of your Reveal. It is the small box on the top of the bridge.
2. Pinch the lower section of the camera.
3. Pull the bottom section upwards, rotating the camera towards you.
4. Point the camera towards your subject (school board, landscape, etc.).
5. When you are done, simply rotate the camera back into its initial position.

Note: The camera can be rotated sideways when it is used for distance viewing.

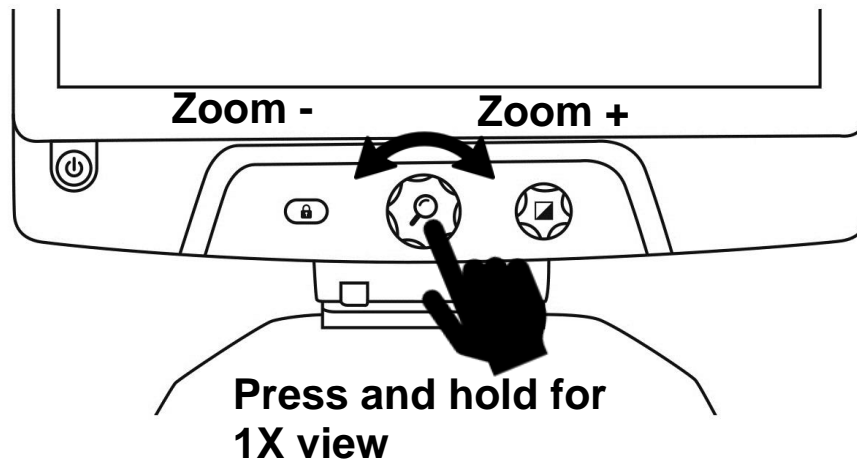
ZOOMING IN AND OUT

The center button can be used to zoom in and out of the image.

Turn the Zoom button clockwise to zoom in.

Turn the Zoom button counterclockwise to zoom out.

When zoomed in, **press and hold on the Zoom button** to get a temporary 1X view of your document. **Release the Zoom button** to return to your previous zoom.


CHANGING CONTRAST

Reveal allows you to customize contrast to better suit your needs and preferences.

Turn the Contrast button to change contrast colors.

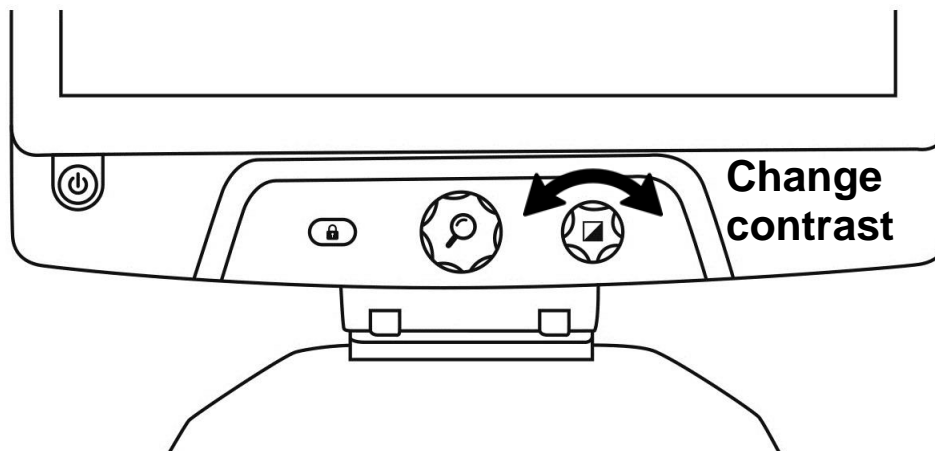Keep turning the button until you find a contrast you are comfortable with.


The default contrast modes are color, black on white, white on black, black on yellow, and yellow on black. More contrasts are available in the menu.

ADJUSTING THE CONTRAST RATIO

The “intensity” of the contrast, or contrast ratio, can be adjusted to your preferences.

To adjust the contrast ratio:

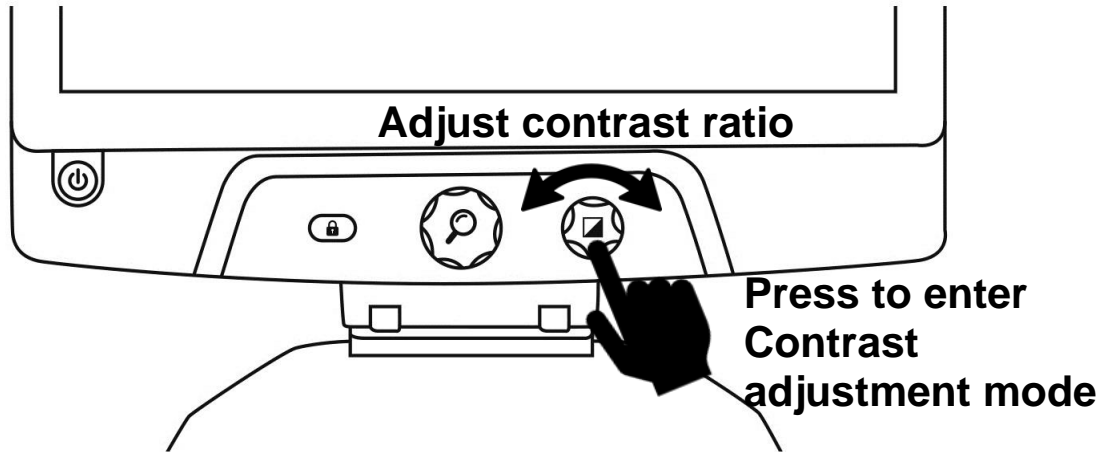1. **Press the Contrast button.** You will now be in contrast adjustment mode.
2. **Turn the Contrast button** counterclockwise to lower the ratio, or clockwise to increase it.
3. **Wait 2 seconds** to return to the normal operation mode.


AUTOFOCUS LOCK

Reveal includes an autofocus system that will constantly attempt to find the best focus possible. Despite this, you always have the option to enable or disable the camera’s autofocus. This is useful when you are performing a manual task like writing on a piece of paper and want to prevent your device from refocusing on your hand or pencil.

To lock the autofocus, press the Autofocus Lock button once. Press again to unlock it. Autofocus will also unlock when changing the zoom level.

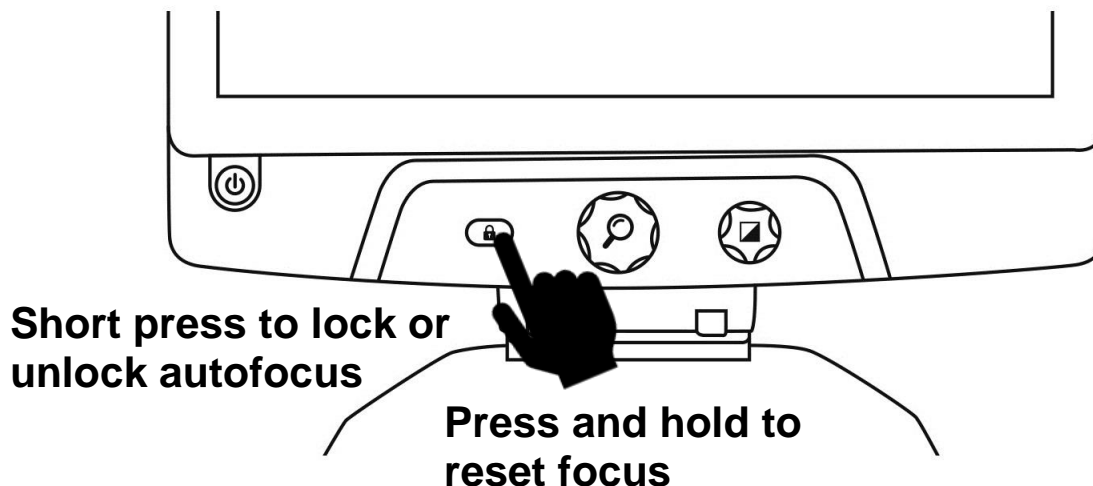
Press and hold the Autofocus Lock button to reset the focus.

The  icon will appear on the top right of your screen when autofocus is locked.

MENU

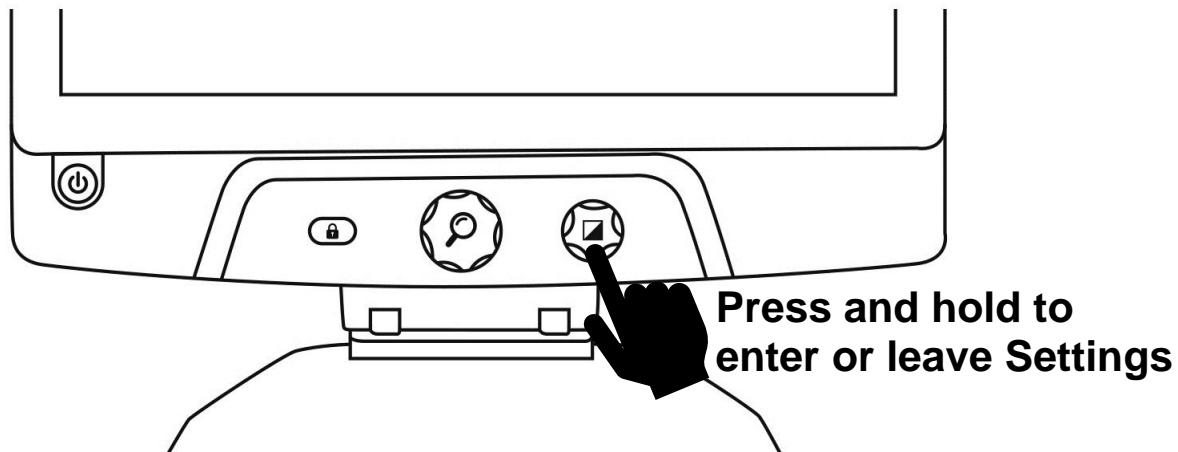
To enter the menu, press and hold the Contrast button.

To navigate the menu, turn the Zoom button.

To select an item or enter a submenu, press on the Zoom button.

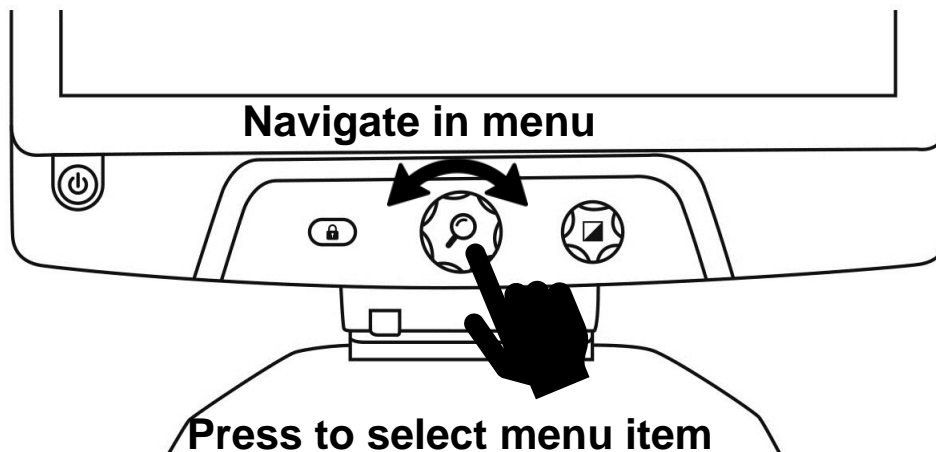To go back to the previous menu, press Contrast, or press and hold Contrast to return to the live camera view.

The menu includes the following items:

Line and blinds: Add a line or blinds on the image for easier reading. Select the adjust menu item to adjust the position on screen.

Screen Brightness: Used to select your preferred screen brightness level.

Enabled Contrasts: Select contrasts available when turning the Contrast button.

Menu Colors: Choose the color scheme for the menu, popup menus and icons.

Preferred Zoom: Choose the zoom level your device starts in.

Auto-off: Choose how long the device stays on when not in use.

Language: Select your device language.

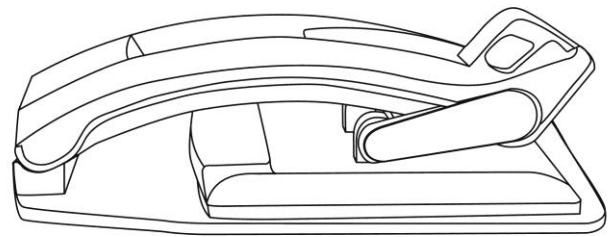
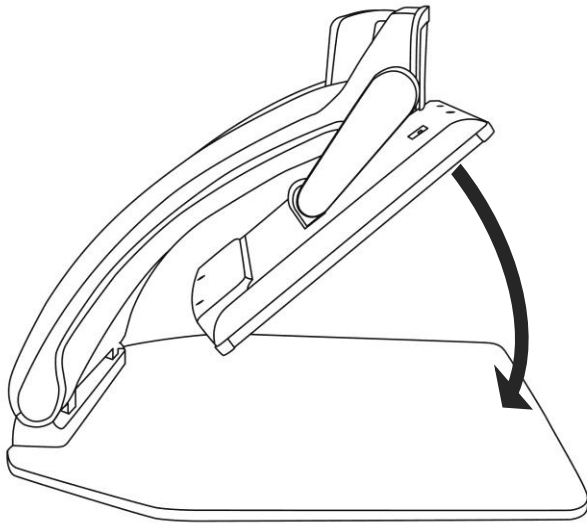
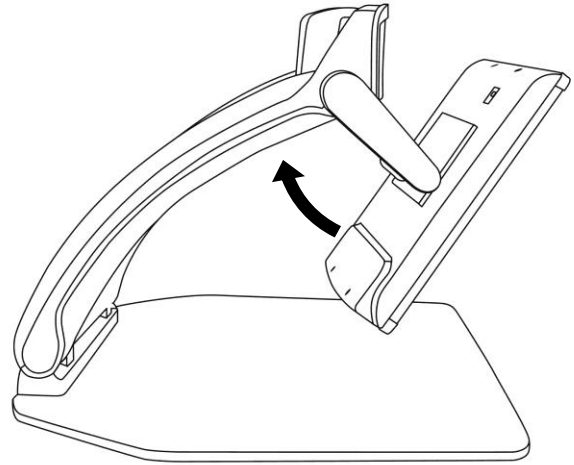
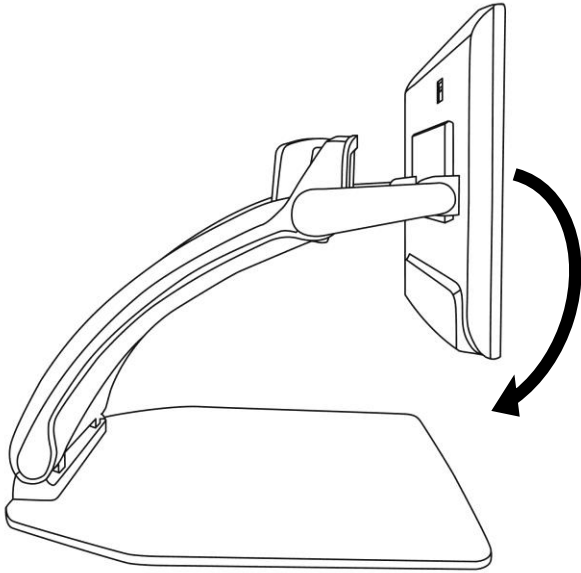
Display: Select the display frequency of your Reveal.

Factory Defaults: Restore all settings to their factory defaults.

About: Information about your Reveal.

STOWING YOUR REVEAL

To stow your Reveal for later use, simply pack it as shown below.

LINE AND BLINDS

You can add a horizontal line or blinds over your image for easier reading. See images below for an example.

Lorem ipsum dolor
 sit amet, consectetur
 adipiscing elit. Mae
 porttitor congue m

Line

sit amet, consectetur
 adipiscing elit. Mae

Blinds

ADDING A LINE OR BLINDS

To add a line or blinds to the screen:

Press the Autofocus Lock and Zoom buttons simultaneously to add/remove a line and blinds.

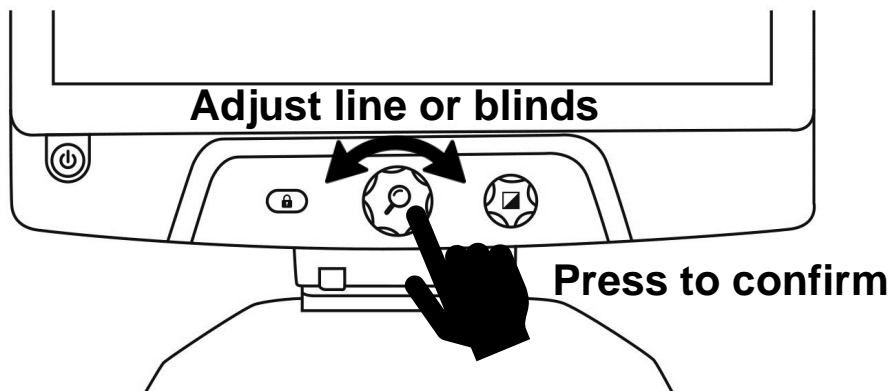
Alternatively, to add a line or blind as your default settings:

1. Open the menu by **pressing and holding the Contrast button.**
2. **Press the Zoom button** to enter the Line and blinds submenu.
3. Select between None, Line, or Blinds according to your preferences.
4. **Press the Zoom button.**

ADJUSTING A LINE OR BLINDS

After selecting a line or blinds, a new “Adjust” option will appear in the menu.

1. Select the adjust option to change the position of the line or the width of the blinds.
2. **Press the zoom button.**
3. **Turn the Zoom button** until you find the perfect position or width.

USING REVEAL ON ANOTHER SCREEN

For a larger image, you may want your Reveal's content to be displayed on an external screen (like a TV). To do so, simply connect your Reveal to an external screen using a High-Resolution video cable.

TROUBLESHOOTING

Reveal does not power on:

Press and hold **Power**. If the unit still does not turn on, make sure your device is plugged in the outlet.

Image is out of focus in magnifier:

Press and hold the Autofocus Lock button or move your hand back and forth over the document you are viewing. This will make your Reveal refocus on the document.

Screen is black:

Make sure the camera is in its default position, aiming at the reading tray.

Reveal shuts down after inactivity:

This is a normal behaviour intended to preserve your screen. By default, Reveal will shut down after 10 minutes of inactivity. It is possible to configure the default auto-off timer to 10 minutes, 20 minutes, 30 minutes, or never.

Reveal does not respond:

Press the **Power** button for 10 seconds. The system will shut down. Then, press Power for one second to turn your Reveal back on.

Other problems:

Restore to factory defaults in the menu.

Press and hold **Contrast**.

Scroll down to Factory Defaults and select Yes.

Warning! All your previous configurations will be lost!

SAFETY AND MAINTENANCE

Do not stare at the LED light source located under the magnifier.

Keep away from water or high humidity. Do not submerge.

Power off your Reveal 16 before cleaning it.

Only use the cables provided with your Reveal 16.

Battery (Optional):

If the Reveal 16 has an optional internal rechargeable Li-Ion battery, the battery must be replaced by a qualified HumanWare technician.

- Do not use or leave the unit near a heat source or fire. Do not store in high temperatures.
- Do not disassemble or modify the unit.
- Do not immerse the unit in water, and do not let it get wet.
- Do not strike or throw the unit.
- Do not pierce or strike the unit with sharp objects or a hammer.
- If the battery leaks and liquid gets into the eyes, do not rub eyes. Instead, rinse the eyes with clean running water and seek immediate medical attention to prevent injury.
- If the battery leaks and comes into contact with skin, immediately rinse area with clean running water to prevent injury.

Cleaning the LCD screen:

Make sure your device is off and unplugged.

Make sure you only use microfiber cloths to clean the LCD screen.

Carefully and gently wipe the microfiber cloth across the LCD screen.

If you choose to clean your LCD screen with a liquid cleaner, make sure this cleaner is appropriate for LCD screens. Distilled water, isopropyl alcohol and a vinegar-water solution (no more than 10% vinegar) are safe cleaning liquids for your LCD screen.

Use only the minimal amount of liquid to prevent spills inside the device. If liquid seeps in, damages to the device might occur. Make sure no liquids remain on your screen. Repeat the previous steps for spots that are harder to remove.

Never use household cleaners to clean your Reveal 16.

Never spray any liquid directly on your LCD screen.

Never use cleaning products that contain ethyl alcohol, ethyl acid, ammonia, acetone or methyl chloride.

Never use a paper towel to clean your LCD screen.

SPECIFICATIONS

Size (unfolded): 370 x 380 x 464 mm

Size (folded): 150 x 380 x 473 mm

Weight (no battery): 5.9kg

Weight (with battery): 6.4kg

Display:

Size: 15.6 inches

Resolution: 1980x1080

Camera zoom:

1-10x (Optical)

1-16x (Digital)

1-45x (Total)

Power Adapter:

Input: 100-240V(50/60Hz), 2.0A

Output: 19VDC, 2.5A

Battery Autonomy: 7 hours

Charge Time: 3.5 hours (when device is off)

Supported High-Resolution video output format:

1920x1080p 60Hz,

No support of DVI emulation

Operating conditions:

Temperature: 10 °C to 40 °C / 50 °F to 104 °F

Humidity: 20% to 80%

Transport and storage conditions:

Temperature: -20 °C to 60 °C / -4 °F to 140 °F

Humidity: 10% to 95%

FCC INFO

FCC Statement:

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Warning:

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in

accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Note: This unit was tested with shielded cables on the peripheral devices. Shielded cables must be used with the unit to ensure compliance.

WARRANTY

HumanWare warrants Reveal, effective from the date of purchase, to be free of any defects in material and workmanship for a period of one (1) year unless otherwise required by law in the country or region of purchase. This warranty applies for a period of ninety (90) days in the case of the optional battery.

This warranty is non-transferable and applies to all cases where the damage is not a result of improper use, mistreatment, or negligence. Improper use is use of this device other than described in this manual. In no event shall HumanWare or its distributors be liable for indirect or consequential damages.

No replacement or repair covered by the warranty will be carried out unless the system is accompanied by a copy of the original bill of purchase. Please keep your original receipt. If the system has to be returned, please use the original packaging.

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