

What's New in Prodigi for Windows v1.2

We're excited to bring you the latest enhancements to Prodigi for Windows, designed to improve your experience and expand functionality! Here's what's new in version 1.2:

More Natural Voices & Greater Choice

For reading documents, Prodigi for Windows now supports a range of installed SAPI5 voices*, including:

- Microsoft desktop voices
- Ivona TTS for Welsh
- Qfrenzy TTS for South African dialects

We've also added and updated the Acapela voices, now powered by Neural TTS. This results in smoother, more lifelike speech, offering a more engaging and comfortable reading experience. Select from new voices like Melany (Canadian English), Vidhi (Indian English), Melanie (Canadian French), and Piotr (Polish). If you previously used the English voice "Saul," you'll be prompted to choose a replacement, ensuring a seamless transition.

Enhanced Reading Experience

Our improved Page Reading Mode lets you enjoy native document and Diamond Edge™ views while listening to text-to-speech. This means more control over how text is displayed and read, allowing you to choose the view that best meets your comfort and visual needs.

Improved Document Management

Need to edit a document on the fly? You can replace, insert, or append pages directly within your Prodigi documents. This added flexibility saves you time and keeps your workflow smooth, whether reorganizing documents, updating content, or adding new material.

Performance and Stability Upgrades

- If another application is using your camera, you'll receive a clear, helpful message—no more guesswork.
- We've removed unnecessary validation steps for quicker application start-up times.

- PDF imports will no longer stall on blank pages, ensuring a more consistent reading experience.
- Enjoy smoother scrolling when reading without TTS.
- Improved compatibility and connection stability for Kodak distance cameras.
- Clearer user messaging when using “Speech ON” for a more intuitive interface.

Additional Enhancements

We’ve also implemented multiple small fixes and refinements to ensure Prodigy remains as user-friendly, efficient, and reliable as ever.

How to Get This Update

When you log in, Prodigy will automatically alert you to the new v1.2 update. Alternatively, you can open Prodigy and go to **Settings > System > Software Update** to check for and apply the latest version at your convenience.

Support Videos

Visit the [Prodigy for Windows YouTube playlist](#) or click the link inside the Help application to learn more about the software’s features, functions, and benefits.

Thank you for choosing Prodigy for Windows. We’re committed to continuously improving your reading and magnification experience. [Share your feedback!](#)

**Most SAPI 5 voices will work with Prodigy for reading documents.*