



Monarch Knowledge Levels

Updating Monarch offline

To update Monarch offline, download the update zip file from either APH or HumanWare support page.

Once downloaded, extract the zip file. You will find two files, as an example:

- Appcast.xml
- Ota_update_xxxxx.zip (xxxx will contain numbers)

1. Copy both of these files to the root directory of a USB thumb drive and insert it into the Monarch.
2. Monarch will automatically detect the update files and prompt you to confirm installation.
3. If you accidentally skip this step, you can manually check for updates by navigating to All Applications, selecting KeyUpdater, and choosing either Check for Updates or Application Update.