

What's New in Prodigi for Windows v1.4

We're excited to share the latest enhancements to Prodigi for Windows, designed to enhance your experience and expand its functionality. Here's what's new in version 1.4:

Prodigi AI Assistant

A new Prodigi AI Assistant is built into the Magnifier application, allowing users to ask questions about any material placed under their document camera. It can recognize printed text, handwriting, math problems, and graphics, and respond with summaries, key points, descriptions, and more. Quick questions are available to help users get started, and settings allow for adjustments like speech mode and voice. Users receive a clear message before any image is shared, and they can navigate between their camera view and the AI responses. Lastly, a Settings Feature Lock can be used to block access to the Assistant when needed. The AI Assistant also includes a daily question counter informing users of their remaining messages each day.

Books Application

Version 1.4 introduces a new Books application, giving users access to full-text content from 16 libraries and 8 magazine providers, including Bookshare, Project Gutenberg, CELA, and others. Users can browse, search, and download books, then read them using their preferred visual or audio settings. The app supports page, line, and column reading modes, as well as bookmarks and page navigation.

Miscellaneous Updates:

- Auto-scroll when reading without TTS has been improved for a smoother experience.
- Fixed an issue where very large file loading could cause a software crash.
- Audio has been enhanced to remove a random “pop” sound when navigating in Magnifier.
- Trial software is the default download and can be activated to the full product by key or invitation.
- New software email templates updated.
- Implemented a long click or finger press to view long file names fully.
- Updated End User License Agreement.

Known Issues:

- In Books:
 - Some content providers requiring OAuth login are not yet supported.
 - Audiobooks are not included in this version.

How to Get This Update

Prodigi will automatically alert you to the new v1.4 update when you log in. Alternatively, you can open Prodigi and go to **Settings > System > Software Update** to check for and apply the latest version at your convenience.

Support Videos

Visit the [Prodigi for Windows YouTube playlist](#) or click the link inside the Help application to learn more about the software's features, functions, and benefits.

Thank you for choosing Prodigi for Windows. We're committed to continuously improving your reading and magnification experiences. [Share your feedback!](#)