

About: Kodak SL10 or SL25 Distance Camera and Connect Tablets

You don't need any additional apps for Kodak camera installation in Prodigy software. Your Connect is compatible with both camera models, but you can only pair one camera at a time. Make sure to use Prodigy software version 1.5.1, or higher, when using Kodak cameras.

Instructions: Wireless Pairing Kodak Cameras to Connect Tablets

1. Charge the camera initially.
2. Turn on the Kodak camera by pressing the power button, and the lens will deploy.
3. On your Connect tablet, navigate Prodigy: Settings → System → Wi-Fi Setup → Wi-Fi Networks.
4. Look for Kodak PIXPRO-SL10_XXXX "network" (no internet required).
 - A. For SL25 model, select PIXPRO-SL25_XXXX.
 - B. XXXX represents unique characters distinguishing cameras of the same make but on a unique channel.
 - C. *IF* you do not see your camera, tap "Scan for Wi-Fi."
5. Tap on the camera "network" being paired. Then tap "Connect".
6. Enter the pairing code: 12345678 and tap Connect to pair.

***Important:** Some newer cameras have 00000000 as their password. This is indicated by a very prominent label.*
7. The Distance application will automatically be added to the Prodigy carousel.

Note 1: If a Sony QX camera was previously installed on the Connect tablet, uninstall the Sony PlayMemories app **before** pairing a Kodak SL camera.

Note 2: Only one camera can be used at a time, even if many are paired. Power on the required camera for the session and turn off the second camera.

Note 3: HumanWare provides special software for Kodak cameras. Off-the-shelf cameras won't work with Connect devices.

Note 4: To enable video functionality, install the Kodak PixPro application and insert a USB card to save files. Connect devices do not support video capabilities.