



Prodigi software

powered by Humañware

Prodigi for Windows Accessibility Guide

Software Accessibility Features	
Functionality	Description
Accessibility Setup	Direct access to Windows accessibility settings to configure mouse pointer color and size, text cursor appearance, and more. Settings → System → Accessibility.
Alternative Access Devices	Supports navigation with keyboard shortcuts, mouse, touchscreen, and alternative access devices like switch scanning.
Autofocus	Automatically focuses on documents or objects, with the option to lock or unlock focus. Double-tap or click in Magnifier (live).
Customizable Color Contrasts	Users can select 24 color combinations and contrasts to suit their visual preferences. Settings → User Interface.
High-Contrast Viewing	Enhances visual clarity and comfort by making text and images more readable for those with low vision. Shift between color, positive, and negative views by swipe or click-drag on the right of the screen.
Highlighted Reading	Displays a positive-negative box around each spoken word while using text-to-speech to help maintain focus. Magnifier and Files → Settings.
Locator Color	An onscreen visual box will indicate the focus of the software. 8 available colors. Settings → User Interface.
Magnification	Provides a broad range of magnification levels (up to 28x for live images and up to 80x for captured images). Magnifier, Distance, and Files.
Multi-lingual Support	Users can install up to four voices from over 75 available in 18+ languages. Additionally, the software interface can be shown in the chosen language. Settings → User Interface.
Optical Character Recognition (OCR)	Recognizes text in captured or imported documents to display in Diamond Edge™ font with user-chosen contrasts.
Pointer Size	Adjust the onscreen pointer on a scale from 1 to 15. Settings → User Interface.
Preferred Zoom	Users can set a default zoom level for captured documents and images. Magnifier and Files → Settings.

Reading Modes	Offers different reading modes (Page, Column, Line) to adapt to various reading preferences and tasks. Magnifier and Files → Settings.
Screen Brightness	Allows users to adjust screen brightness to suit their visual needs and lighting conditions. Settings → User Interface.
Sounds	Audio feedback is provided for specific actions like clicking an application. Settings → Audio.
Speech Settings	Options include no speech, only for document reading, or for everything in the software's interface. Magnifier → Settings and Settings → Audio.
Text-to-Speech (TTS)	Convert printed text into spoken words, providing access to printed materials. Adjustable in Settings and Files.
User Interface	Three major accessibility features can be adjusted: 1) the number (1, 2, or 4) of application icons that show in the Carousel, 2) the available applications that show in the Carousel (AI Assistant✓, Books✓, Distance, or Apps), and 3) full screen or Windows-mode Settings → User Interface → Interface.
Voice Selection	Allows users to choose from multiple text-to-speech voices in different languages and dialects to suit their preferences. Settings →, User Interface, and Files.

Keyboards (when using Prodigy)

Standard	HumanWare Bluetooth (New Windows – April 2026)	Result
Ctrl + C	Ctrl + C	Copy
Ctrl + X	Ctrl + X	Cut
Ctrl + V	Ctrl + V	Paste
Windows logo key + Up arrow	NA	Maximize Window
Windows logo key + M	NA	Minimize Application
Windows logo key + Tab	NA	Open Task View
Windows logo key + D	NA	Display and Hide Desktop

Alt + Tab	Alt + Tab	Switch between open Apps
Windows logo key + X	NA	Open Quick Link menu
Windows logo key + L	NA	Lock your PC
Ctrl + Q	Ctrl + Q	Quit
ESC	ESC	Back
F1	F1	Open settings for current view
F2	F2	Open AI Assistant application
F3	F3	Open Magnifier application
F4	F4	Open Distance application
F5	F5	Open Books application
F6	F6	Open Files application
F7	F7	Decrease reading speed
F8	F8	Capture image / Start/Stop reading
F9	F9	Increase Reading Speed
F10	F10	Decrease volume
F11	F11	Mute or unmute
F12	F12	Increase volume
Arrow Keys	Arrow Keys	Move Locator up, down, left, right; scroll
Ctrl + Up Arrow	Ctrl + Up Arrow	Pan in a captured image Scroll text in column reading mode Split-screen changes app window focus
Ctrl + Down Arrow	Ctrl + Down Arrow	Pan in a captured image Scroll in column reading mode Split-screen changes app window focus

Ctrl + Left Arrow	Ctrl + Left Arrow	Pan in a captured image Scroll text in line reading mode Scroll Left/Previous Page Change app window focus in split-screen
Ctrl + Right Arrow	Ctrl + Right Arrow	Pan in a captured image Scroll text in line reading mode Scroll Right/Next Page Change app window focus in split-screen
Plus (+)	Plus (+)	Zoom in / Increase text size
Minus (-)	Minus (-)	Zoom out / Decrease text size
Ctrl + B	Ctrl + B	Show/Hide Button Banner
ESC Ctrl + Q Alt + F4	ESC Ctrl + Q Alt + F4	Exit the current application
Ctrl + S	Ctrl + S	Save image
TAB Shift + TAB	TAB Shift + TAB	Move locator focus
Shift + F10	Shift + F10	Open context menu of an item
Enter Space	Enter Space	Confirm a selection or an action A long 1-second press opens context menu of an item Start/stop reading

Touchscreens

Gesture	Result
Pinch and Zoom	Slide two fingers together to zoom out, spread them apart to zoom in.
Tap	Tap once to select applications, preferences, etc.

Double Tap	Double tap twice quickly: In Carousel header, exits to Windows. In Magnifier app to enable/disable autofocus.
Long Press	Press and hold for about 1 second: In Magnifier to capture images. In Files to open action menu. Open context menu of selected item
Swipe	Up, down, left, and right to navigate. In Magnifier, swipe up or down on left side of screen to toggle between Live and Multi-Capture Modes. In Magnifier or Files, swipe up or down on right side of screen to cycle contrasts: Color, Positive, Negative.
2-finger Long Press	Expand full text of selected item (menu or file name)

Mouse

Action	Result
Left-click	Select an item on screen (application, menu item, file, etc.)
Right-click	In Files, open context menus for delete, rename, etc.
Click-hold (~1 sec.)	In Magnifier, capture an image In a file starts/stops reading
Scroll wheel	In Magnifier, zoom in and out In Settings (or other menus), navigate up and down Move locator
Click-drag	In Magnifier, on the left side of screen toggles between Live and Multi-Capture Modes In Magnifier or open file, toggles between Color, Positive, and Negative contrasts.
Double-click	In Magnifier, enables/disables auto-focus. In a file, enables/disables speech.

[Let us know how we are doing!](#)