

Prodigi

Quick Start Guide

Welcome to Prodigi for Windows

All Prodigi accounts begin with a trial version to help you get started quickly.

Follow this guide to **download**, **install**, and **activate** the software for full access.

.....

1



Prodigi software

powered by Humañware

Download and install



<https://qrco.de/Prodigi>

2

Terms and Conditions



Decline

Accept

3



**Your Prodigi trial
expires in 14 days**

Continue Trial

Activate

Contact Us

Purchase

Choose the option that applies to you

Invitation Organizational User

4a

← I Have an:

Invitation

Activation Key

Existing Account

5a

Invitation

name@example.com

6a

You have received
an invitation

Skip

Decline

Accept

7a

Create Account

First name

Last name

Email (user name)

name@example.com

Country



Next

8a

Create Account

Password



Confirm password



Back

Sign up

Activation Key Individual User

4b

← I Have an:

Invitation

Activation Key

Existing Account

5b

← Activation Key →

Email

Activation Key



6b

Activate Account

Email (user name)

name@example.com

Activation key

Password



Confirm password



Cancel

Activate

Go to Step 9

Existing User 2nd Device

4c

← I Have an:

Invitation

Activation Key

Existing Account

5c

Sign in

Email (user name)

Next

6c

Sign in

name@example.com

Password



[Forgot password?](#)


Next

Go to Step 9

9

← Language	Voice	Speech	Usage Data?
Selected voices: 0 of 4	Darius (US) >	On - Direct Touchscreen	Yes
English >	Deepa >	On - Alternative Access	No
Other languages... >	Other voices... >	Documents only	Details >
		Off	

10

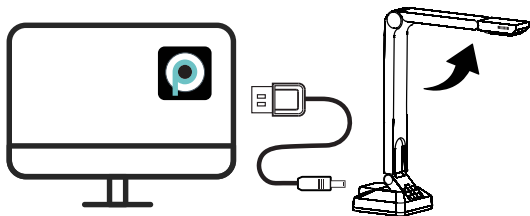


11

← Camera
Integrated Webcam
HoverCam Solo 8 Plus
Other camera...

Magnifier

12

**Start
magnifying**

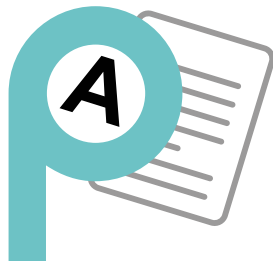

For more information on how to use Prodigy for Windows,
please refer to the **User Guide**.

www.humanware.com/support/prodigi_software/