Prodigi software

powered by HumanWare

Cleaning Instructions for the Prodigi All-in-One

The Prodigi All-in-One combines an integrated Windows PC and monitor with a HoverCam Solo 8+ document camera, a wired keyboard, and a wired mouse. Because this unit is shared between patients in a clinical setting, follow the general steps below to clean and disinfect it between each use.

Before you begin

- Power off the system completely, or put the monitor to sleep per your facility's protocol, before cleaning.
- Avoid touching cables, ports, or the power connector with a wet cloth or wipe.
- Have on hand an EPA-registered disinfectant wipe suitable for electronics (or a soft cloth lightly dampened with mild soap and water), plus a dry microfiber cloth.

All-in-One PC and monitor

- Wipe the housing, stand, and base with a disinfectant wipe or a cloth dampened with mild soap and water.
- Clean the screen with a soft microfiber cloth. If using a disinfectant wipe on the screen, choose one labeled safe for electronics and displays, and avoid letting liquid pool along the bezel edges.
- Never spray liquid directly onto the screen or housing. Apply it to the cloth or wipe first, then wipe the surface.
- Avoid ammonia-based glass cleaners and abrasive materials, which can damage anti-glare coatings.
- Let all surfaces air dry fully before powering the unit back on or inviting the next patient to use it.

HoverCam Solo 8+ document camera

- Wipe the camera's arm, base, and housing with a disinfectant wipe or a cloth dampened with mild soap and water.
- Clean the camera lens only with a dry, soft microfiber cloth, using gentle circular motions from the center outward. Keep liquid away from the lens.
- Never submerge the camera, spray it directly, or use paper towels or abrasive materials on the lens, as this can scratch the lens coating and affect image quality.
- Keep moisture away from the USB connector.

Keyboard and mouse

- Wipe the top surfaces, keys, mouse housing, and cables with a disinfectant wipe or a cloth dampened with mild soap and water.
- Wring out wipes or cloths that are dripping before use, and avoid excess moisture that could seep between keys or into the mouse housing.
- Let both dry fully before the next use.

Between patients: high-touch points

- Prioritize the keyboard, mouse, monitor bezel, and camera arm and base, since these are the surfaces touched most often.
- Follow the disinfectant label's recommended contact or dwell time before wiping the surface dry, if the product calls for one.
- Allow all surfaces to dry completely before the next patient uses the unit.

What to avoid

- Ammonia, bleach, or glass cleaners on the screen or camera lens.
- Spraying cleaner directly onto any component. Apply it to a cloth or wipe first.
- Submerging any component in liquid, or using ultrasonic cleaners.
- Paper towels, tissues, or abrasive cloths on the screen or camera lens.
- Cleaning the unit while it is powered on and in use.

Note: The Prodigi All-in-One includes components manufactured by third parties (AOC and HoverCam). The steps above reflect general practices commonly used for cleaning shared computer and camera equipment in a clinical setting. They do not replace instructions, warranty terms, or safety guidance published by the component manufacturers, and they do not override your facility's own infection control protocols. Where manufacturer guidance and facility protocol are available, follow those first.

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