Cleaning Instructions for the Prodigi Complete Kit

The Prodigi Complete Kit includes a Microsoft Surface Pro tablet, a HoverCam Solo 8+ document camera, a wireless keyboard and mouse, and a docking station. Because this kit is shared between patients in a clinical setting, follow the general steps below to clean and disinfect it between each use.

Before you begin

- Power off the Surface Pro completely, or put it to sleep per your facility's protocol, before cleaning.
- Disconnect the Surface Pro from the docking station and unplug charging cables where practical.
- Avoid touching cables, ports, or the charging connector with a wet cloth or wipe.
- Have on hand an EPA-registered disinfectant wipe suitable for electronics (or a soft cloth **lightly dampened** with mild soap and water), plus a dry microfiber cloth.

Microsoft Surface Pro

- Wipe the back panel, edges, and kickstand with a disinfectant wipe or a cloth dampened with mild soap and water.
- Clean the screen with a soft, dry microfiber cloth. If a wipe is needed, use one labeled safe for electronic displays and avoid letting liquid pool along the screen edges.
- Never spray liquid directly onto the screen or body. Apply it to the cloth or wipe first, then wipe the surface.
- Avoid ammonia-based glass cleaners and abrasive materials, which can damage the screen's anti-reflective coating.
- Keep moisture away from the charging port, USB-C port, and speaker openings.
- Let all surfaces air dry fully before powering the unit back on, reconnecting it to the dock, or inviting the next patient to use it.

HoverCam Solo 8+ document camera

- Wipe the camera's arm, base, and housing with a disinfectant wipe or a cloth dampened with mild soap and water.
- Clean the camera lens only with a dry, soft microfiber cloth, using gentle circular motions from the center outward. Keep liquid away from the lens.
- Never submerge the camera, spray it directly, or use paper towels or abrasive materials on the lens, as this can scratch the lens coating and affect image quality.
- Keep moisture away from the USB connector.

Wireless keyboard and mouse

- Wipe the top surfaces, keys, and mouse housing with a disinfectant wipe or a cloth dampened with mild soap and water.
- Wring out wipes or cloths that are dripping before use, and avoid excess moisture that could seep between keys or into the mouse housing.
- Remove batteries before cleaning if it makes it easier to reach the battery compartment seams, and let both dry fully before reassembling and returning to use.

Docking station

- Power down or disconnect the dock from the Surface Pro before cleaning, and unplug it from wall power if practical.
- Wipe the exterior housing with a disinfectant wipe or a cloth dampened with mild soap and water.
- Keep liquid away from all ports (USB, HDMI, Ethernet, SD/microSD card slot) and the power connector.
- Let the dock dry completely before reconnecting cables or the Surface Pro.

Carry bag

- Wipe exterior surfaces and handles with a disinfectant wipe as needed, following the care guidance on the bag's label, and let dry before repacking equipment.

Between patients: high-touch points

- Prioritize the keyboard, mouse, Surface Pro screen and kickstand, dock ports, and camera arm and base, since these are the surfaces touched most often.
- Follow the disinfectant label's recommended contact or dwell time before wiping the surface dry, if the product calls for one.
- Allow all surfaces to dry completely before the next patient uses the kit.

What to avoid

- Ammonia, bleach, or glass cleaners on the screen or camera lens.
- Spraying cleaner directly onto any component. Apply it to a cloth or wipe first.
- Submerging any component in liquid, or using ultrasonic cleaners.
- Paper towels, tissues, or abrasive cloths on the screen or camera lens.
- Cleaning any component while it is powered on and in use, or while connected to the dock.

Note: The Prodigi Complete Kit includes components manufactured by third parties (Microsoft, HoverCam, Logitech, and the docking station manufacturer). The steps above reflect general practices commonly used for cleaning shared computer and camera equipment in a clinical setting. They do not replace instructions, warranty terms, or safety guidance published by the component manufacturers, and they do not override your facility's own infection control protocols. Where manufacturer guidance and facility protocol are available, follow those first.

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