Cleaning Instructions for the Reveal 16 and Reveal 16i

The Reveal 16 and Reveal 16i are desktop magnifiers with a 15.6-inch screen, a camera on a folding bridge, and a reading tray that forms the base of the unit. Both models are sold with and without an X-Y table. Because these units are shared between patients, clean and disinfect them between each use. Steps that apply to only one model are marked.

In the steps below, a disinfectant wipe means an EPA-registered wipe made for electronics. A soft cloth dampened with mild soap and water works just as well.

Before you begin

- Turn the unit off and unplug it. Remove the power cable and any video cable. Reveal 16i: also unplug anything in the USB port or the audio jack.
- Reveal 16i: the screen is a touchscreen. Wiping it while the unit is on acts like a finger and can change settings.
- Leave the unit open on a steady surface so you can reach the screen, the bridge, the camera, and the tray.
- Have ready a dry microfiber cloth for the screen and lenses, a second microfiber cloth for damp wiping, and a disinfectant wipe for every other surface.
- Safe liquids for the screen are distilled water, isopropyl alcohol, or water mixed with no more than 10 percent vinegar. Use as little as possible so none seeps inside.

Screen

- Wipe the screen with a dry microfiber cloth. Use microfiber only.
- If a dry wipe is not enough, dampen the cloth with one of the safe liquids above. Wipe in one direction rather than scrubbing, and keep liquid off the edges and the bezel.
- Isopropyl alcohol on a microfiber cloth both cleans and disinfects. If you use a pre-moistened wipe instead, check that the label says it is safe for screens.
- Reveal 16: the screen is a display only, so there is no reason to press on it.
- Reveal 16i: every patient touches this screen, so treat it as a high-touch surface. Fingerprints and skin oil are the usual soil. Hold the screen by its edges, support the back with your free hand, and do not press hard.

Housing, bridge, and screen mount

- Wipe the housing, the back of the screen, and the bridge with a disinfectant wipe. The bridge is the folding arm that holds the camera over the tray.
- Include the sides of the screen and the tilt and height adjustment area, since staff and patients grip the screen there.
- Wipe along the hinge, not into it. Keep moisture out of the fold seam and out of the point where the bridge locks open.

Buttons and dials

- These are the most touched surfaces on the unit. The Reveal 16 has four controls: Power, Autofocus Lock, Zoom, and Contrast. The Reveal 16i has five: Power, Autofocus Lock, Switch, Zoom, and Contrast/Volume.
- Wipe each control with a disinfectant wipe, then dry it with a microfiber cloth.

- The last two controls on both models turn and also press. Wipe around their base rather than working liquid into the gap, because moisture underneath can affect how they respond.

Cameras and LED light

- The Reveal 16 has one camera on the bridge. The Reveal 16i has three: two in the bridge head, plus a front camera beside the screen.
- Clean each lens with a dry microfiber cloth only. Wipe in small circles from the center outward and keep all liquid off the lenses.
- Reveal 16i: the front camera picks up the same fingerprints as the touchscreen, so clean it whenever you clean the screen.
- Wipe the camera housing with a disinfectant wipe, keeping liquid off the lenses.
- The bridge camera turns for distance viewing. Point it back at the tray before you clean, and hold it by the housing rather than the lens edge.
- Wipe around the LED light with a barely damp cloth, keeping liquid off the light itself.

Reading tray

- The reading tray is the flat base of the unit. On units without an X-Y table, patients rest documents here, so treat it as a high-touch surface.
- Wipe the tray and its front edge with a disinfectant wipe. Leave the disinfectant on as long as the label says, then wipe the surface dry.
- Keep liquid away from where the bridge meets the tray, and from the underside of the tray, where the connectors are.

X-Y table, on units so equipped

- Both models are sold with and without the table. If the unit has no table, clean the reading tray instead and skip this section.
- Lock the table so it does not slide while you work. This takes a firm grip, so brace the unit with your other hand.
- Wipe the top of the table and its raised edges with a disinfectant wipe. After the controls, patients touch this surface most.
- Wipe the sides, then unlock the table, slide it through its full travel, and wipe what was hidden underneath. Lock it again when you are finished.
- Keep liquid out of the mechanism under the table, and never oil or grease the tracks.
- Brush crumbs, eraser debris, and paper scraps off with a dry cloth or a soft dry brush.
- Let the table dry, then check that it still slides and locks normally.

Ports, power, and battery

- Keep every port and connector dry. Wipe only the plastic around them, with a barely damp cloth.
- Reveal 16i: this includes the USB port and the audio jack. The micro-USB port is for service only, so leave it alone.
- Reveal 16i: keep liquid away from the speaker grilles and the microphone on the front. These openings lead straight inside the unit.

- Do not open the battery compartment. Only a qualified HumanWare technician may replace the battery.
- Plug the power cable back in only after every surface is dry.

Carrying case and accessories

- Wipe the outside of the carrying case or rolling bag with a disinfectant wipe, and let it dry before you repack the unit.
- Wash or replace the microfiber cloths when they look soiled. A dirty cloth will scratch the screen and the lenses.
- With the adapter unplugged, wipe its body and cable, keeping moisture off the connector ends.

Between patients

- Focus on the controls, the sides and bezel of the screen, the X-Y table or reading tray, and the front edge of the tray. Reveal 16i: add the touchscreen and the front camera.
- Let every surface dry before the next patient uses the unit. Fold the unit only once it is dry, or you will trap moisture between the screen and the tray.

What to avoid

- Household cleaners of any kind.
- Ethyl alcohol, ethyl acid, ammonia, acetone, or methyl chloride. These damage the screen and lens coatings.
- Spraying liquid onto any part of the unit. Put it on the cloth first.
- More liquid than you need, since the excess can seep inside.
- Paper towels, tissues, or rough cloths on the screen or a lens.
- Compressed air, cotton swabs, or picks in the ports, around the dials, or in the X-Y table mechanism.
- Soaking any part of the unit, or using it near water or in damp air.
- Cleaning the unit while it is on, in use, or plugged in.
- Folding or packing the unit before it is completely dry.

Note: These instructions do not replace the safety guidance or warranty terms in the Reveal 16 and Reveal 16i user guides, or your facility's own infection control protocols. Where those apply, follow them first.

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