Cleaning Instructions for the explorē 5, explorē 8, and explorē 12

The explorē 5, explorē 8, and explorē 12 are handheld and portable electronic magnifiers built around an integrated screen and camera. Because these devices are shared between patients in a clinical setting, follow the general steps below to clean and disinfect them between each use. Steps that apply to only one or two models are noted.

Before you begin

- Power off the device completely, or put it to sleep per your facility's protocol, before cleaning.
- Disconnect the device from its charging cable and unplug it from wall power where practical.
- Avoid touching the charging port or video output port with a wet cloth or wipe.
- Have on hand an EPA-registered disinfectant wipe suitable for electronics (or a soft cloth lightly dampened with mild soap and water), plus a dry microfiber cloth. Each device ships with its own microfiber cloth.

Screen and housing (all models)

- Wipe the housing, back panel, and edges with a disinfectant wipe or a cloth dampened with mild soap and water.
- Clean the screen with a soft, dry microfiber cloth. If a wipe is needed, use one labeled safe for electronic displays and avoid letting liquid pool along the screen edges or bezel.
- Never spray liquid directly onto the screen or housing. Apply it to the cloth or wipe first, then wipe the surface.
- Avoid ammonia-based glass cleaners and abrasive materials, which can damage the screen's coating.
- Keep moisture away from the USB-C charging port and video output port.
- Let all surfaces air dry fully before powering the unit back on or inviting the next patient to use it.

Buttons and touch controls

- explorē 5, 8, and 12: Wipe the large control buttons with a disinfectant wipe or a cloth dampened with mild soap and water, taking care not to let moisture seep around the button edges.
- explorē 8 and explorē 12: The touchscreen doubles as the primary control. Clean it using the screen guidance above, and avoid pressing hard enough to scratch the touch surface.

Integrated camera(s)

- Clean the camera lens only with a dry, soft microfiber cloth, using gentle circular motions from the center outward. Keep liquid away from the lens.
- Wipe the area surrounding the lens with a disinfectant wipe or a cloth dampened with mild soap and water, keeping liquid away from the lens itself.
- explorē 8 and explorē 12 each have two integrated cameras; clean both lenses using the same method.
- Never submerge the device, spray cleaner directly onto the camera, or use paper towels or abrasive materials on the lens, as this can scratch the lens coating and affect image quality.

Units, reading stands and handle

- Wipe the stand's built-in (explorē 5, 8, and 12) and external (explorē 8 and 12) surfaces and folding hinges with a disinfectant wipe or a cloth dampened with mild soap and water.
- explorē 5, 8, and 12: Keep moisture away from the stand's built-in LED lighting elements.
- Let the stand dry completely before folding it closed or repacking it in the carrying case.

Carrying case and accessories

- Wipe exterior surfaces of the carrying cases with a disinfectant wipe as needed and let dry before repacking equipment.
- Launder or replace the included microfiber cleaning cloth and screen protector per your facility's protocol if visibly soiled.

Between patients: high-touch points

- Prioritize the screen, buttons or touchscreen surface, housing edges, and camera lens area, since these are the surfaces touched or viewed most often.
- Follow the disinfectant label's recommended contact or dwell time before wiping the surface dry, if the product calls for one.
- Allow all surfaces to dry completely before the next patient uses the device.

What to avoid

- Ammonia, bleach, or glass cleaners on the screen or camera lens.
- Spraying cleaner directly onto any component. Apply it to a cloth or wipe first.
- Submerging any component in liquid, or using ultrasonic cleaners.
- Paper towels, tissues, or abrasive cloths on the screen or camera lens.
- Cleaning the device while it is powered on and in use, or while connected to a charger.

Note: The explorē 5, explorē 8, and explorē 12 are HumanWare products. The steps above reflect general practices commonly used for cleaning shared handheld and portable electronic devices in a clinical setting. They do not replace instructions, warranty terms, or safety guidance published by HumanWare, and they do not override your facility's own infection control protocols. Where manufacturer guidance and facility protocol are available, follow those first.

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